



EZ-ONE

ONE FOR ALL




For Better Endo

EZ-ONE


ONE FOR ALL




ISO color coding

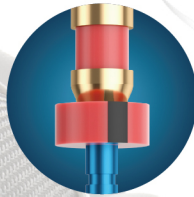

radiopaque stopper

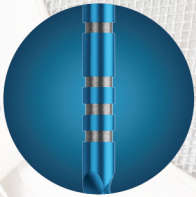

precise depth marking

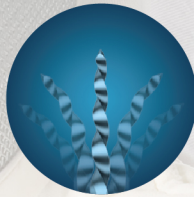


R-phase NiTi



parallelogram




rotation

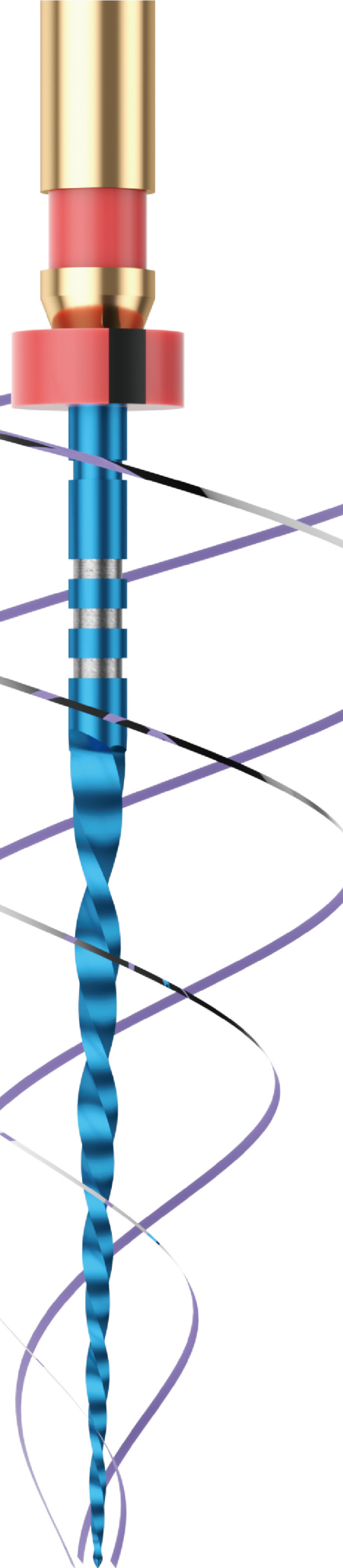

Features

- MaxTech 6.0 NiTi alloy
- Proper flexibility and strong cutting ability
- One file acts as shaper and finisher
- More space for debris removal
- Regressive taper to bring accurate cutting
- Progressive pitch reduce screw-in
- Single file technique

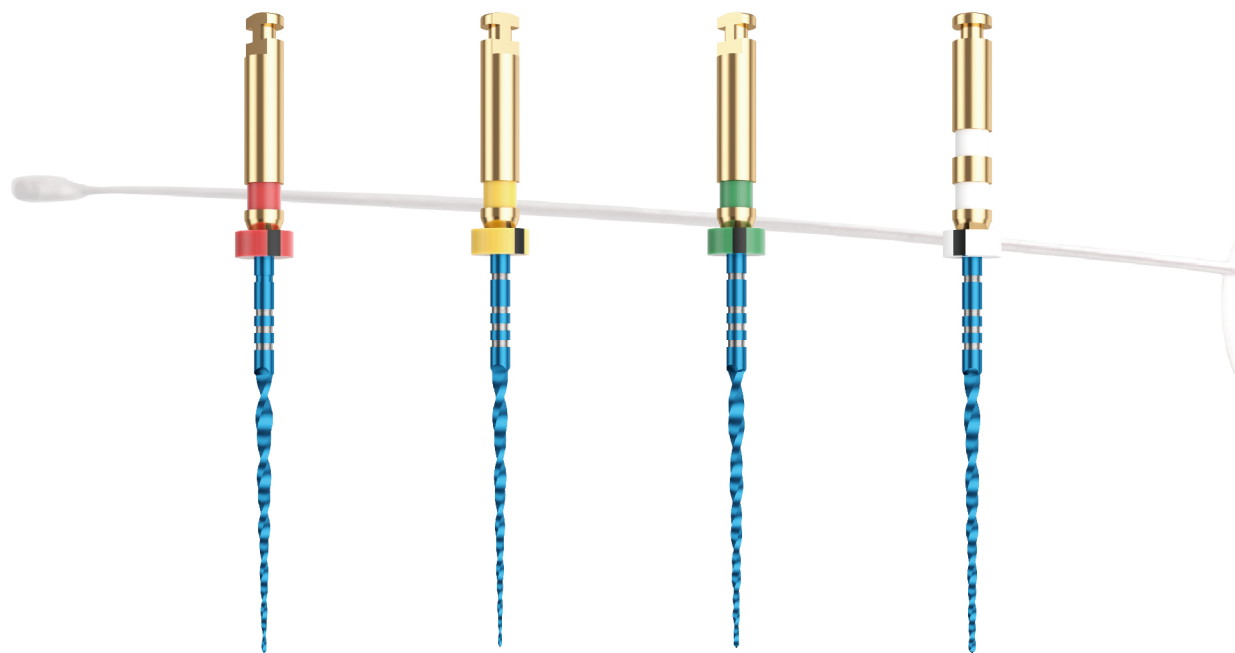

| | | | | |
|--------------|---------|--------|--------|--------|
| EZ-ONE | Primary | Small | Medium | Large |
| Size | 25/.07 | 20/.07 | 35/.06 | 45/.05 |
| Speed(RPM) | 400 | 400 | 400 | 400 |
| Torque(N.CM) | 3.0 | 1.5 | 1.5 | 3.0 |

Sequence

- Scout the canal with hand use file
- Create glide path by any path function file
- Choose proper size shaping files
- Shaping the canal with the chosen EZ-ONE until it reaches working length
- After shaping and finishing is done, then choose the matched GP points for obturation.

SSK #10

T-Blue

Primary



◆ Small 20/.07 for narrow canals

◆ Primary 25/.07 for most cases

◆ Medium 35/.06 & Large 45/.05
for wide canals

Small
20/.07

Medium
35/.06

Large
45/.05





References

| | | | 19mm | 21mm | 25mm | 31mm |
|--|---------------------------------------|----------------------|--------------|--------------|--------------|--------------|
| EZ-ONE single size 3pcs/blister | Small | 20/.07 | | EZO 021 2007 | EZO 025 2007 | EZO 031 2007 |
| | Primary | 25/.07 | | EZO 021 2507 | EZO 025 2507 | EZO 031 2507 |
| | Medium | 35/.06 | | EZO 021 3506 | EZO 025 3506 | EZO 031 3506 |
| | Large | 45/.05 | | EZO 021 4505 | EZO 025 4505 | EZO 031 4505 |
| EZ-ONE assorted sizes 3pcs/blister | Basic Kit (SX- Blue, T-Blue, Primary) | 19/.04 17/.02 25/.07 | | EZO 021 A001 | EZO 025 A001 | EZO 031 A001 |
| | Advanced Kit (Small, Primary, Medium) | 20/.07 25/.07 35/.06 | | EZO 021 A002 | EZO 025 A002 | EZO 031 A002 |
| EZ-CAN single size 3pcs/blister | SX- Blue | 19/.04 | BFU 019 1904 | | | |
| | T- Blue | 17/.02 | | BFT 021 1702 | BFT 025 1702 | BFT 031 1702 |

Clinic Cases



Endo Motor & Obturation



Primary
60 points
Color coded
NIC® CE 0197
Gutta Percha Points
Size: Primary
⚠ For Professional Dental Use Only

Small: GPW 029 020
Primary: GPW 029 025
Medium: GPW 029 035
Large: GPW 029 045

Primary
100 points
Sterilized
Color coded
NIC® CE 0197
Absorbent Paper Points
Size: Primary
⚠ For Professional Dental Use Only

Small: PPW 029 020
Primary: PPW 029 025
Medium: PPW 029 035
Large: PPW 029 045



Premium manufacturer
since 2002

SUPERLINE



SHENZHEN SUPERLINE TECHNOLOGY CO., LTD.

ISO 13485:2016 CE 197 FDA

www.supline.com