

# Instruction For Use

## Dental Root-Canal Instruments

### Paste Carriers

Document No:SLT-CE-01-002

Issue date:2021-06-26

Version:D5

<b>Compiled By</b>	Tao Li	
<b>Audited By</b>	Jinsong Zhou	
<b>Approved By</b>	Zhimin Li	
<b>Revision Date</b>	2021-06-26	

# Dental Root-Canal Instruments

## Instruction For Use

### Paste Carriers

**<Product Name 产品名称>** : Dental Root-Canal Instruments 根管器械

**<Type/model 类型/型号>** : Paste Carriers 输送针

**<Specifications 规格>**: 25,30,35,40;

#### **<Intended use 预期用途>**

Root canal filling or drug delivery.

对根管填充或药物输送。

#### **<Structure and composition 结构和组成>**

It is mainly composed of working part and rod.

主要由工作部分和杆组成。

Main materials 主要材料:

Working part: stainless steel.

工作部分: 不锈钢

Handle: copper alloy.

手柄: 铜合金。

#### **<Contraindication 禁忌症>**

The product contains nickel or chromium and should not be used in people who are allergic to this metal.

该产品含有镍或铬，不得用于对这种金属过敏的人。

#### **<Warning 警告>**

1. This product can only be used by professional dentists;

本产品只允许专业牙医使用；

2. Ensure that the product is disinfected every time it is used;

确保每次使用时都要对该产品消毒；

3. Do not use this product for any purpose other than dental treatment and service.

请勿使用本产品除牙科治疗和服务以外的用途。

### **<Precautions for use 使用注意事项>**

1. For your own safety, please wear personal protective equipment (gloves, glasses, mask);

为了您自己的安全，请穿戴个人防护设备（手套，眼镜，口罩）；

2. Check whether the product is within the specified period of validity before use;  
使用前检查产品是否在规定的有效期；

3. To prevent infection, please disinfect the product with autoclave before use;  
为防止感染，请在使用前用高压灭菌器对产品进行消毒；

4. Select the most appropriate specification for each case;  
为每种情况选择最合适的规格；

5. The allowable speed range of root canal motor was 250-1000 rpm;  
根管马达允许的转速范围为 250-1000rpm；

6. This product is one-time use and is forbidden to be used twice.  
本产品为一次性使用，禁止 2 次使用。

7. This product should be treated as medical waste.  
本产品在处置时应视为医疗垃圾。

### **<Usage method 使用方法>**

The paste was attached to the surface of the delivery needle ,and the needle was rotated to deliver the paste into the root canal.The operation was repeated until the whole root canal was filled with paste.

将糊剂附着在输送针表面，旋转输送针往根管里面输送糊剂，重复操作，直至整个根管塞满糊剂。

### **<Cleaning, disinfection and sterilization 清洗、消毒、灭菌>**

1. After opening the package, clean the product with sodium hypochlorite.  
打开包装后，将产品用次氯酸钠清洗；

2. Disinfect the product with an approved and effective disinfectant solution.  
使用批准有效的消毒溶液对产品进行消毒；

3. The product was packed with sterilization bag and sterilized by high temperature and high pressure steam at 134°C for 18 minutes.

产品使用灭菌袋包装后进行高温高压蒸汽灭菌，温度 134℃，时间 18min；

#### <Product storage conditions 产品储存条件>

1. During transportation and storage, handle with care, do not stack, to prevent damage to packaging materials;

运输存储过程中轻拿轻放，请勿堆叠，防止损坏包装材料；

2. Avoid storage in high temperature, high humidity and direct sunlight, and store the products at room temperature;


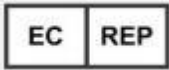





避免存放在高温、高湿和阳光直射的地方，产品要在室温下存放；

#### <Validity 有效期>

The product is valid for 5 years.

产品有效期 5 年。

#### <Symbol identification 符号标识>

			
Manufacturer	European representative	Non sterile	See directions for use
制造商	欧盟代表	没有灭菌	见使用说明书
			\
LOT number	Date of manufacture	Validity	\
批号	生产日期	有效期	

#### <Version or date 版本或日期>

The release date of the instructions for use:2021-06-26.

使用说明书的发布日期：2021-06-26。



SHENZHEN SUPERLINE TECHNOLOGY CO., LTD.

Company Address: Room 302, Nanshan Zhiyuan A4, No. 1001 Xueyuan Avenue,  
Changyuan community, Taoyuan Street, Nanshan District, Shenzhen, 518000  
Guangdong, P.R. China

Factory Address: B401 & 501, Haomai Industry Park, No. 2 Rongfeng Road, Jinsha  
Community, Kengzi Street, Pingshan, Shenzhen 518122, Guangdong P.R. China  
Tel: (+86) 755-26016056, 26016156;

Fax: (+86) 755-26016026;

Mail: [slt@supline.com](mailto:slt@supline.com).

Web: [www.supline.com](http://www.supline.com).

深圳市速航科技发展有限公司

公司地址: 深圳市南山区桃源街道长源社区学苑大道1001号南山智园A4栋第3层302房


工厂地址: 中国广东省深圳市坪山坑梓街道金沙社区荣峰路2号豪迈工业园 B401 及 501  
, 邮编: 518122;

电话: (+86) 755-26016056, 26016156;

传真: (+86) 755-26016026;

邮箱: [slt@supline.com](mailto:slt@supline.com)

网址: [www.supline.com](http://www.supline.com)

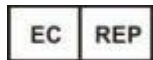

European representative: Wellkang Ltd

Address: The Black Church, St. Mary's Place, Dublin 7, D07 P4AX, Ireland.

Tel: +353(1)4433560

Fax: +353(1)6864856

Web: [www.CE-marking.com](http://www.CE-marking.com)